

ALOCLAIR® PLUS GEL - INFORMATION LEAFLET

Aloclair Plus Gel works by forming a protective film in the oral cavity that covers the exposed nerve endings of the lesion (aphthas) thus avoiding further irritation and reducing pain. The hyaluronic acid (high molecular weight) and the Aloe vera in the formulation support the natural healing of damaged tissues.

Aloclair Plus Gel is indicated for aphthous ulcers, aphthous stomatitis, minor lesions, chafing and traumatic ulcers caused by braces and ill-fitting dentures and lesions of surgical origin. Aloclair Plus Gel is suitable for the use in adults and children of all ages.

Instructions for use:

Apply 1 or 2 drops of gel so as to cover the whole mouth ulcer or lesion. Avoid direct contact of the applicator with the lesion. Do not touch the lesion with the tongue for at least 2 minutes so as to allow for the protective film to form. Close the cap after each application. Use 3-4 times a day, or as needed and avoid eating or drinking for at least 1 hour after application.

Ingredients:

Aqua, polyvinylpyrrolidone (PVP), maltodextrin, propylene glycol, PEG-40 hydrogenated castor oil, xanthan gum, potassium sorbate, sodium benzoate, sodium hyaluronate, aroma, benzalkonium chloride, disodium EDTA, sodium saccharin, dipotassium glycyrrhizate, aloe barbadensis.

Cautions:

Each Aloclair Plus Gel tube should be used in a single patient to avoid cross contamination of the applicator and gel. The use of Aloclair Plus Gel is contraindicated in patients with a known history of hypersensitivity to any of the ingredients.

Precautions and warnings:

Do not use the product if the package appears damaged. Store at room temperature, away from direct heat. Do not freeze. Keep out of the reach of children.

If you frequently suffer from mouth ulcers seek medical advice.

Contents: 8ml e

Manufacturer:

Alliance Pharma S.r.l.
Viale Restelli 5, 20124 Milano, Italy

Aloclair®, Alliance and associated devices are trademarks of the Alliance Pharma PLC group.